

# DILAPAN-S® PRODUCT INFORMATION

## How Supplied

Dilapan-S is available in  
boxes of 25 rods

Rods come in the following dimensions:

- **4 mm x 65 mm:** 25 ea/bx
- **4 mm x 55 mm:** 25 ea/bx
- **3 mm x 55 mm:** 25 ea/bx



## Storage and Handling



Store between  
+15°C and +30°C.

**No refrigeration needed**



Keep away from direct  
sunlight and high humidity.



Do not freeze.

## Distributors



**Online Portal Registration Form:**

<https://www.hpsrx.com>

**Customer Service:** 800-850-1657

**Fax:** 800-361-6984

**Email:** [customerservice@hpsrx.com](mailto:customerservice@hpsrx.com)



(via 3PL/Specialty Pharmaceutical Services)

**Contact Customer Service  
for Ordering Form**

**Customer Service:** 844-985-1368

**Fax:** 614-652-9760

**Email:** [GMB-SPS-Medicem@cordlogistics.com](mailto:GMB-SPS-Medicem@cordlogistics.com)



TO LEARN MORE ABOUT **DILAPAN-S**

• Visit [www.DilapanS.com](http://www.DilapanS.com) • **Medical inquiries**, email: [usmedical@medicem.com](mailto:usmedical@medicem.com) • Contact your **Dilapan-S Sales Representative**

Please see additional Important Safety Information on the following page and full [Instructions for Use](#).

# Important Safety Information

## Indications for Use

Dilapan-S® is for use by healthcare professionals trained in OB/GYN whenever cervical softening and dilation are desired, such as cervical ripening during term labor induction or gynecological procedures that require cervical preparation.

## Contraindication:

Dilapan-S is contraindicated in the presence of clinically apparent genital tract infection.

## Warnings & Precautions:

- Dilapan-S is intended for single use only. **Do not** reuse, resterilize, reprocess, or use if primary packaging has been opened or damaged. Discard after use.
- Careful placement of the device is essential to avoid traumatic injury to the cervix or uterus (see [Instructions for Use—Insertion](#)). The device should not be left in place more than 24 hours. **Instruct patients to:** Report any excessive bleeding, pain, or temperature elevation, and to avoid bathing, douching, and intercourse. Patients should return to the physician for removal of Dilapan-S at the indicated time and should be instructed not to attempt self-removal under any circumstances.
- Potential Complications/Risks: Twisting of device during removal may cause the device to break (see [Instructions for Use—Removal](#)). Complications may include: device entrapment and/or fragmentation, expulsion, or retraction; patient discomfort or bleeding; spontaneous rupture of membranes; spontaneous onset of labor; cervical laceration.